



EYELET HOOK KIT



- (2) Eyelet Hooks
- (2) 5/16" screws
- (2) 1/4" screws
- (2) 3/8 screws
- (2) washers

www.saebo.com/saebostep

1

Select screw length that is right for your shoe.



2

Insert proper screw and washer into eyelet on your shoe.



3

Stabilize the head of the screw with your finger while securely twisting the Eyelet Hook with other hand.



4

Position the open end of the Eyelet Hook toward the front of the shoe.



5

Attach Front Cord to Eyelet Hooks as shown.



6

Wind the dial by turning clockwise until the Front Cord is tight and you have the lift and angle desired.



 **Saebo, Inc.**

2459 Wilkinson Blvd., Suite 120-B
Charlotte, NC 28208

888-942-5751
sales@saebo.com
www.saebo.com

EC REP

European Healthcare & Device Solutions (Ireland) Ltd.
info@europeandevicesolutions.co.uk
www.europeandevicesolutions.co.uk

Australian Sponsor:
Ausmedic Australia Pty Ltd
Unit 3, 3 Basalt Road Pemulwuy NSW
2145 1300 473 422
Performhealth.com.au

WARNING
This product can expose you to Chloroform/Phthalate (COP),
which is known to the State of California to cause cancer.
For more information go to WWW.P65Warnings.ca.gov

CE MD  